

MyAmie pendant

What is the MyAmie?

The MyAmie is a small and discreet pendant designed to provide more comfortable wearing options for the user. It is compatible with Tunstall's range of Lifeline home units and other Tunstall telecare enabled systems. The MyAmie operates on the 869MHz dedicated social alarm frequency, enabling maximum reliability, and providing the wearer with increased peace of mind and reassurance that they have the means to call for help whenever they should need it.

The MyAmie pendant is waterproof to IP67 standard and can be worn in the shower, and also incorporates an LED to provide the user with visual reassurance that the alarm button has been pressed.

How does it work?

The MyAmie pendant enables the user to raise a call for help in their home or garden, simply by pressing the large red button within a 50m range of the Lifeline home unit. The MyAmie will then send a radio signal to the Lifeline home unit which will in turn raise a call to the monitoring centre.

The MyAmie offers a variety of easy to change wearing options:

- **Wrist strap** - soft hypoallergenic, elasticated strap with clasp available in a choice of 2 widths
- **Neckcord** - with safety break link
- **Belt clip** - allowing the user to attach the pendant to their clothing. It can also be used to wall mount the pendant

- **Easy press adaptor** - an option for people with limited dexterity
- **Key ring** - option is provided to enable the pendant to be attached to keys

Who is it for?

The MyAmie is ideal for:

- Older, infirm or disabled people
- Individuals who have been discharged from hospital and require additional support and assistance at home
- People at risk of domestic violence, racial harassment, repeat victimisation or distraction burglary
- Those living in high crime areas
- People of any age living alone

Why is it needed?

The MyAmie has been developed following customer feedback showing an increasing number of people preferring to wear the pendant on their wrist. After extensive testing, Tunstall has designed a very comfortable to wear pendant that is half the size of the Amie+, yet the red button is a third larger. As a result users will be more likely to wear the pendant, providing them with additional protection as their ability to raise an alarm call is increased.




All the reassurance you need

Tunstall

Features and benefits

- 50% smaller*
- 30% bigger button size*
- 65% lighter*
- Designed to be more comfortably worn on the wrist
- 2 wrist straps sizes available:
 - 10mm wide provided as standard
 - 20mm wide option provides a longer and wider strap*
- Improved auto low battery monitoring - increased the reliability of auto low battery alerts
- Extended battery life - up to 7 year life depending on usage (previously 5 years)

Wearing options

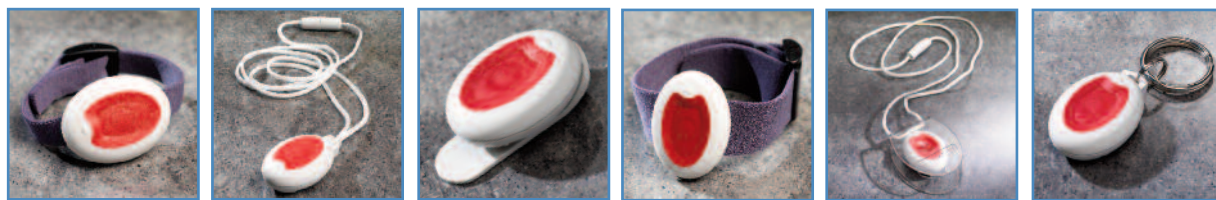

10mm wrist strap

Neckcord

Belt clip

20mm wrist strap

Easy press adaptor

Key ring

Technical details

Radio Frequency:	869.2125MHz
Weight:	7g
Size:	36 x 27 x 14mm (L x W x D)
Battery Life:	Up to 7 years (non replaceable)
Waterproof:	IP67 (Up to 1m for 30mins) They can be worn in the shower however wearing them in the bath should be avoided where possible.
Radio Range:	Up to 50m
Frequency:	869.2125MHz
Compatibility:	All Tunstall 869MHz alarm systems

Standards

CE:	
EMC:	EN55022, ETSI EN301 489-1, ETSI EN301 489-3
Safety:	EN 60950
Radio:	ETSI EN300 220-2
Social Alarm:	EN50134-2, EN50130-4

Part number:

<i>MyAmie³</i>	<i>P68005/01</i>
<i>Wrist Strap 20mm (pack of 10)</i>	<i>D6702145A</i>
<i>Wrist Strap 10mm (pack of 10)</i>	<i>D6702135A</i>
<i>Belt Clip (pack of 10)</i>	<i>D6702136A</i>
<i>Neckcord (pack of 10)</i>	<i>D6702137A</i>
<i>Easy Press Adaptor</i>	<i>D6702138A</i>
<i>Key Ring</i>	<i>D6702128A</i>

Tunstall

Our policy of continual development means that product specification and appearance may change without notice. Tunstall does not accept responsibility for any errors and omissions contained within this document.

© 2011 Tunstall Group Ltd. © TUNSTALL and LIFELINE are registered trademarks.

Tunstall Healthcare (UK) Ltd, Whitley Lodge, Whitley Bridge, Yorkshire DN14 0HR
Tel: 01977 661234 Fax: 01977 662450 Email: enquiries@tunstall.co.uk

www.tunstall.co.uk

981/03/11

